



The Sunwheels Beach Wheelchair is a cost-effective and lightweight accessibility solution designed to provide safe and easy access to beaches, lakes, and other soft-ground environments for people with reduced mobility.

While not a floating wheelchair, the Sunwheels is water-friendly and can be safely used in shallow water, making it ideal for shoreline access and recreational use.



ADVANTAGES



- > Water-friendly design
- > Excellent mobility on sand and soft terrain
- > Lightweight and portable
- > Tool-free & quick assembly
- > Corrosion Resistant
- > Foldable & space-saving
- > Customizable with optimal accessories
- > Quick-Drying fabric

Technical specifications

Corrosion resistant frame	Galvanized aluminum frame with epoxy powder coating - polyethylene wheels for optimal performance on sand and soft terrain			
Sunwheels dimensions	Width	Length	Height	Weight
	90 cm 35"	125 cm 49"	121 cm 47"	22 kg 48.5 lbs
Seat dimensions	Width	Depth	Height	Weight capacity
	46 cm 18"	39 cm 15"	34 cm 13"	Min: 25 kg 55 lbs Max: 135 kg 300 lbs
Package dimensions & weight	Package dimensions		Weight	Color
	111 x 60 x 38 cm 44" x 24" x 15"		22kg 48,5lbs	Blue & Orange
Rear wheel diameter	44 cm – 17"			
Options	Sun canopy - Transport bag			
Standards compliance	EN12182 CE and EN13485 CE			

Conditions of use

The Sunwheels is specifically designed for use on soft ground by individuals with reduced mobility.

Supervision & Safety

- All users must be accompanied by an attendant at all times.
- The Sunwheels is water-friendly, it is not designed to float but suitable for use in shallow water.
- For safety, users are strongly advised to wear a life jacket when using the chair in water.
- Always comply with local and state regulations regarding life jacket requirements.

Weight Capacity

- Maximum user weight: 135 kg | 300 lbs
- Do not exceed this limit under any circumstances.

Easy installation & low maintenance

QUICK INSTALLATION

The Sunwheels® Beach Wheelchair is designed for fast, tool-free assembly and disassembly in under 5 minutes. Assembly steps:

1. Unfold the frame and ensure it is fully open and stable
2. Attach the backrest securely
3. Install the wheels and lock them into position
4. Mount and adjust the footrest
5. Add optional accessories (sun canopy)

Before use, always verify that all components are properly secured.

EASY MAINTENANCE

After Each Use

- Rinse thoroughly with fresh water
- Allow the wheelchair to dry completely
- Clean using mild soap only

Regular checks : inspect fasteners, wheels, and accessories for wear or damage.

Do not exceed the maximum weight limit of 135 kg (300 lbs)



Endless applications

SAND



OCEAN & SEA



LAKE



SWIMMING POOL



What are our additional products ?

Mobi-mat® Accessibility mat



Mobi-roller™ Mats dispenser



Mobi-com™ Advertising mat



Mobi-walk All-terrain rollator



info@mobi-mat.com