

This user notice refers to Sunwheels, the beach and pool wheelchair.
Please read the following instructions carefully and always follow the safety guidelines.

INSTALLATION

Step 1: Attach the padded backrest bar, padded support, and wheels connections to the frame using the fixing pins. Secure the wheels to the main frame and ensure all tubes are properly locked.

Step 2: Insert the wheels onto the chassis axle. Align the hole in the hub with the hole in the axle, then insert the retaining clip. Make sure each wheel is securely locked in place.

Step 3: Insert the backrest into the frame rings. Insert the two fasteners fully through the appropriate holes to secure it. Ensure the clips are properly locked to the main frame. Then attach the strap located at the lower end of the backrest by wrapping it around the wheel axle.

Step 4: Adjusting the leg rest

Remove the clips. Position the leg rest in one of the three available positions, and reinsert the clips securely.

Step 5: Anti-tip bar

To replace or adjust the anti-tip bar, remove the two clips and pull the bar out. Insert the new bar in the same position and secure it with the clips.

Step 6: Installing the Front (Third) Wheel

Unscrew the two bolts (L) with a 5mm Allen wrench. Separate the tubes. Insert the wheel, reassemble the tubes and tighten the bolts securely.

Step 7: Installing the armrests

Attach the right and left armrest to the backrest and tighten the screws using 6mm Allen key to ensure a secure fit.



CONDITIONS OF USE

The Sunwheels wheelchair is designed for people with reduced mobility on soft ground such as sand or similar terrain. Users must always be assisted by an attendant.

Sunwheels is not a floating wheelchair and is not designed for use in rough water or strong currents.. It can be used in shallow, calm water (e.g., shoreline, lakes, or pool access).

For safety, it is recommended that users wear a life jacket when entering the water.

Please check local and state regulations regarding life jacket requirements.

Weight limit: 135kg / 300lbs

SAFETY INSTRUCTIONS

- Do not use any part of the wheelchair for purposes other than its intended use.
- Do not use the wheelchair components to carry heavy loads or lift the chair.
- Improper use may cause damage or detachment of components, leading to injury.
- Always use the seat belt during operation to enhance user safety.

WARRANTY

Sunwheels® is covered by a 24-month manufacturer warranty against material and manufacturing defects.

The warranty does not cover:

- Normal wear and tear
- Damage caused by overloading
- Misuse or improper use
- Unauthorized modifications or repairs
- Improper storage or maintenance

Any intervention by unauthorized third parties voids the warranty.

REFERENCES

